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1 Usage according to intended purpose

The endoscopy adapter, available as an accessory for full-face masks and consisting of a non-vented retaining ring and an elbow with a reclosable opening, allows the physician to perform endoscopy (bronchoscopy) with continuous ventilation.

2 Safety

Risk of injury if endoscopy is not performed correctly!

Endoscopes may injure the respiratory tract.


⇒ Endoscopy should only ever be performed by trained healthcare professionals.

For safety information, indication, and/or contraindications, please refer to the instructions for use of the mask used and the instructions for use of the endoscope used.

3 Compatibility

Endoscopy adapter	Article no.	Compatible mask
Endoscopy adapter NV, for - JOYCEeasy - JOYCEone - JOYCEclinic	WM 15864	JOYCEone Full Face JOYCEone Full Face NV JOYCEeasy Full Face JOYCEeasy comfort Full Face JOYCEeasy X Full Face JOYCEeasy next Full Face JOYCEeasy next Full Face NV JOYCEclinic Full Face NV JOYCEclinic Full Face NV + AAV (leakage 1) JOYCEclinic Full Face vented (leakage 2) Intensa Full Face NV Intensa Full Face NV + AAV (leakage 1) Intensa Full Face vented (leakage 2)
Endoscopy adapter NV, for LENA	LMT 15968	LENA, LENA NV

4 Description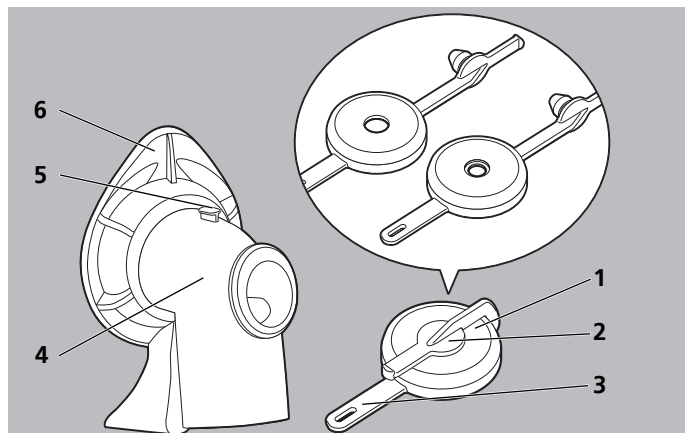


No.	Designation	Description
1	Sealing cap	Seals the opening in the endoscopy adapter.
2	Sealing plug	Seals the opening in the cap.
3	Saddle	Secures the cap in place.
4	Endoscopy adapter	Allows an endoscope to be introduced nasally or orally.
5	Retaining hook	Allows the saddle to be fixed in place.
6	Retaining ring non-vented	Connects the endoscopy adapter to the mask.


5 Preparation and operation

5.1 Fit endoscopy adapter

1. Select cap size.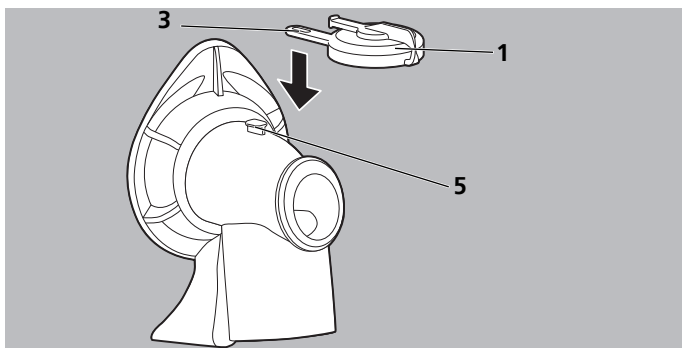


2. Attach saddle 3 of cap 1 to retaining hook 5.

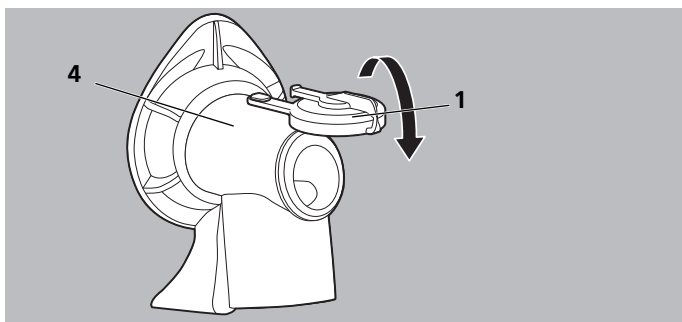
3. Fit cap 1 onto opening of endoscopy adapter 4.

5.2 Introduce endoscope

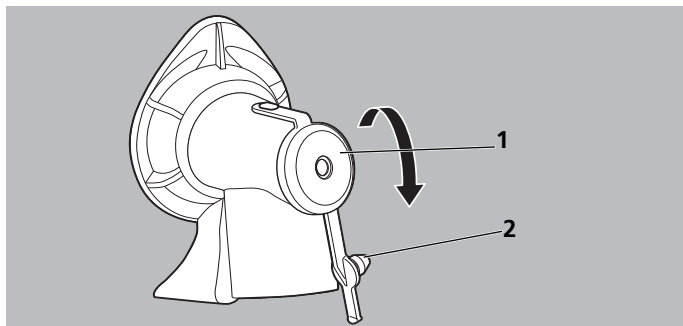
1. Remove sealing plug **2** from cap **1**.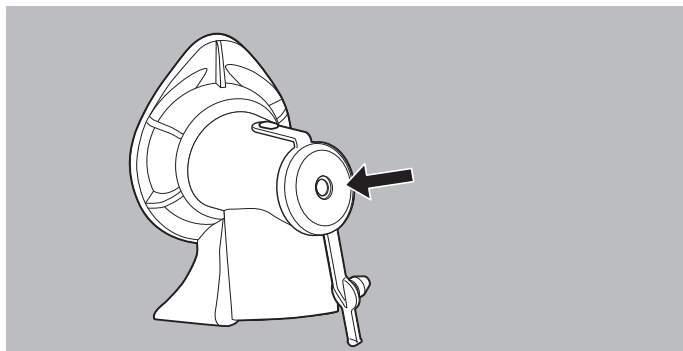

2. Introduce endoscope.

5.3 Endoscopy adapter on mask

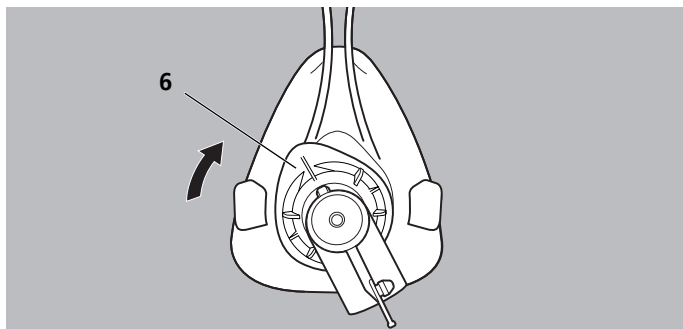1. Insert the retaining ring **6** in the mask body at the 11 o'clock position and turn clockwise to 12 o'clock until it locks into place.

6 Hygiene treatment

This product is a disposable item (2).

Disposable items are intended to be used only once. Therefore use these items only once and do not reprocess them. Reprocessing disposable items may impair the functionality and safety of the product and lead to unforeseeable reactions due to ageing, embrittlement, wear, thermal stress, the effects of chemical processes etc.

7 Disposal

You can dispose of all parts in domestic waste.

8 Appendix

8.1 Technical data

Specification	Product	
Product class to (EU) Medical Device Regulations 2017/745	IIa	
Dimensions (W x H x D) incl. non-vented retaining ring and cap, for JOYCE	50 mm x 60 mm x 75 mm	
Dimensions (W x H x D) incl. non-vented retaining ring and cap, for LENA	45 mm x 55 mm x 58 mm	
Tube connection: Tapered connection as per EN ISO 5356-1 non-vented	Ø 22 mm (female)	
Cap Size 1: Size 2:	Opening: 3.3 mm 4.8 mm	Endoscope: Ø 3.5 mm to 5 mm Ø 5 mm to 6.5 mm
Standards applied	EN ISO 17510: 2020	

8.2 Materials

Only use the mask after consulting your physician if you are allergic to any of the following substances:

Article	Material
Endoscopy adapter seal	(SI) silicone
Retaining ring non-vented, for JOYCE	(PP) polypropylene
Retaining ring non-vented, for LENA	(POM) polyoxymethylene
Elbow for endoscopy adapter	(PC) polycarbonate

8.3 Scope of supply

Part	Item number
Endoscopy adapter NV, for JOYCEeasy/JOYCEone/ JOYCEclinic	WM 15864
Endoscopy adapter NV, for LENA	LMT 15968

8.4 Markings and symbols

Symbol	Description
	Lot number
	Order number
	Manufacturer and, if necessary, date of manufacture

Symbol	Description
CE 0197	CE symbol (confirms that the product conforms to the applicable European directives/regulations)
	Permitted temperature range for transport and storage
	Keep out of sunlight
	Use by date
MD	Indicates the product is a medical device
UDI	Unique device identifier (uniform product code for medical devices)
	Follow the instructions for use

8.5 Warranty

Löwenstein Medical Technology gives the purchaser of a new original Löwenstein Medical product and of a spare part fitted by Löwenstein Medical Technology a limited manufacturer warranty in accordance with the warranty conditions applicable to the product in question and in accordance with the warranty periods from date of purchase listed below. The warranty conditions are available on the manufacturer's website. We will also send you the warranty conditions on request.

Please bear in mind that any claim under warranty and liability shall be void if neither the accessories recommended in the instructions for use nor genuine spare parts are used.

8.6 Declaration of conformity

The manufacturer, Löwenstein Medical Technology GmbH + Co. KG, Kronsaalsweg 40, 22525 Hamburg, Germany, hereby declares that the product complies with the relevant provisions of the Medical Device Regulations (EU) 2017/745. The complete text of the Declaration of Conformity is available on the manufacturer's website.

As a user and/or patient, you must report any severe adverse events occurring in connection with the product to the manufacturer and to the responsible authorities.

8.7 Contact information

Info@azeermed.com

+966551177970 Azeer Customer Support

+966126502921 Azeer Customer Support

www.azeermed.com

P.O. Box 132391 Jeddah 21382 KSA. Prince majed Street

Manufacturer
Löwenstein Medical
Technology GmbH + Co. KG

Kronsaalsweg 40
22525 Hamburg, Germany

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www.loewensteinmedical.com

CE 0197

LÖWENSTEIN
medical

LMT 67496SA0