



MiniScreen Viewer


Installation Instructions



Dear Customer,

You purchased a sleep diagnosis device from us or your device has just been maintained/repared. You will either have received the MiniScreen Viewer software CD together with the device or we will have sent you a download link. You now need to update or install the MiniScreen Viewer software! Please follow the instructions below to do so.

1

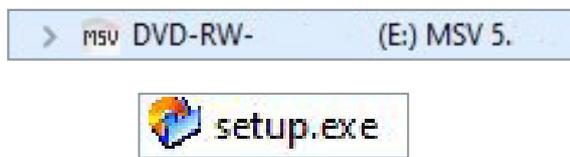


Insert the software CD into your computer's CD drive and double-click to select the installation CD in Windows Explorer. Alternatively you can open the installation file you downloaded for the MiniScreen Viewer software. Double-click **setup.exe** to start the installation.

2




Select your language in the installation assistant and confirm with **Start!**

3

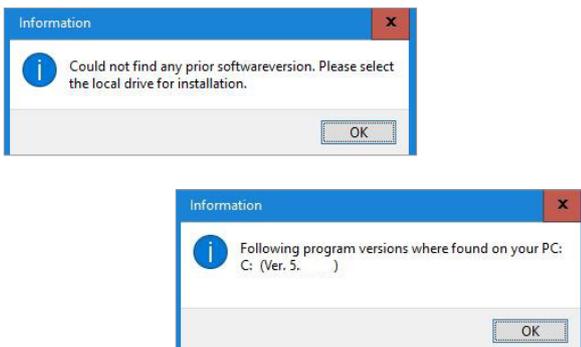


Depending on your selection of "new installation" or "update", one of the two windows shown above will open. Confirm with **OK**.

4

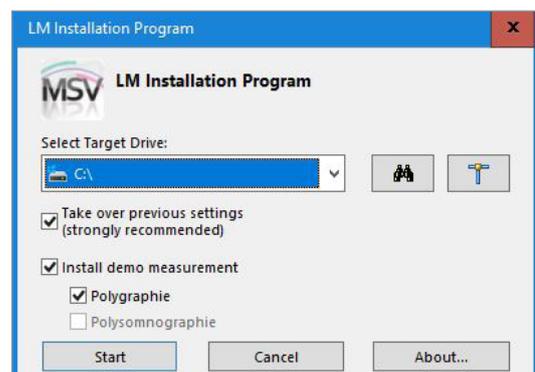

We recommend that you use the standard settings. Click **Start**.

5

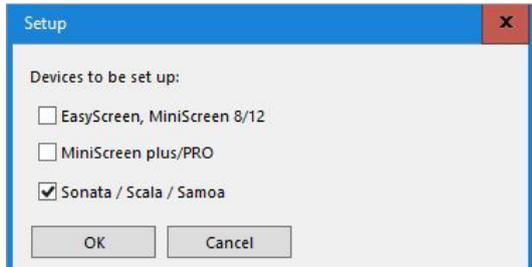



New installation:

Select your device.

Update:

All settings are retained. Confirm your selection with **OK**.

6

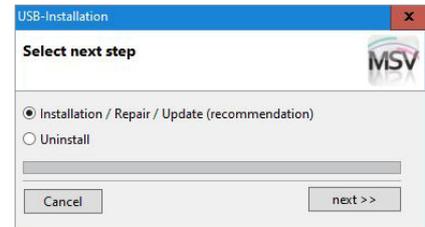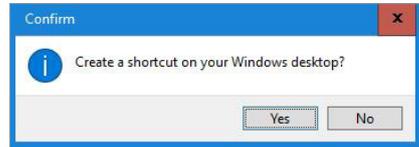

Confirm with **Yes**.



Next select **Installation / Repair / Update** and confirm your selection with **next**.

7



Confirm this dialog box by clicking **Install**.

8

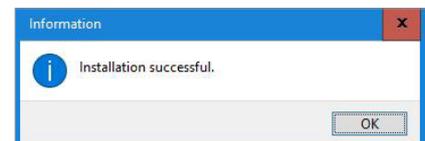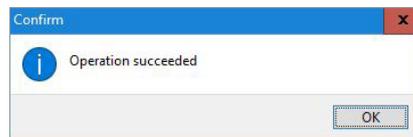

Confirm with **OK**.

If you require any help during the installation process, please contact your local specialist dealer.




 Dr. Fenyves und Gut Deutschland
 Daimlerstraße 23
 72414 Rangendingen, Germany

Sales + Service
 Löwenstein Medical
 Arzbacher Straße 80
 56130 Bad Ems, Germany
 T. +49 2603 9600-0
 F. +49 2603 9600-50
 info@loewensteinmedical.com
 loewensteinmedical.com

