



Hygienic Reprocessing of Löwenstein Medical Ventilators at Change of Patient in Hospitals

VENTIlogic LS · VENTIlogic plus
 prisma VENT30 · prisma VENT40 · prisma VENT50 · prisma VENT50-C
 prisma30ST · prisma30ST-C

Important information:

- Use the ventilator **ONLY** with device bacteria filter or HMEF.
- This notice applies only to hygienic reprocessing and handling of ventilators.
- Observe the manufacturer's instructions for the use of products for cleaning, disinfecting or sterilizing devices.
- When treating accessories, please follow the manufacturer's instructions for use and read information about holding times and cleaning, disinfecting and sterilizing.
- Disposable articles are intended for single use only. Use these articles only one time and dispose of them.
 Do not reprocess for another use.

Procedures for change of patient



Wipe disinfection

of device housing exterior
 (e.g., with terralin® protect)



Replacement

- of bacterial filter or HMEF
- possibly air filter and pollen filter
- of patient circuit or sterilization of reusable circuit system



Function check



If the therapy device was operated without the device bacteria filter or filter (air and pollen filter) or if proof of multi-drug resistant pathogens (MRSA, 3MRGN, etc.) is found, the hygienic reprocessing must be carried out by **the manufacturer or a dealer authorized by the manufacturer!**

